





**KN95**

Advanced Respirator

Low Respiratory Resistance

A comfortable fit



**20** pcs/box

**BFE  $\geq$  95%**



**Coziness**  
**Non-woven fabric**



**Close fit**  
**It fits the face**



**Air permeability**  
**Low Suction Resistance**



## FUNCTION

WEARING TO COVER AND PROTECT THE MOUTH AND NOSE,CAN EFFECTIVELY PREVENT THE INHALATION OF DUST AND DROPLETS.

## USAGE

- 1.FACE THE MASK WITHOUT THE NOSE CLIP,AND THE NOSE CLIP SHOULD BE ON TOP;
- 2.HOLD THE MASK ON YOUR FACE AND AGAINST YOUR CHIN;
- 3.PULL THE STRAPS ON BOTH SIDES AND HANG ON THE EARS,AND ADJUST TILL YOU FEEL COMFORTABLE;
- 4.PUT YOUR FINGERS IN THE MIDDLE OF THE METAL NOSE CLIP,AND MOVE YOUR FINGERTIPS TO BOTH SIDES UNTIL THE NOSE CLIP IS FULLY PRESSED INTO THE SHAPE OF THE NOSE SLOPE;
- 5.PUT YOUR HANDS ON YOUR FACE TO CHECK THE TIGHTNESS WITH YOUR FACE.



## PRODUCT INFORMATION

PRODUCT NAME: PROTECTIVE MASK

PRODUCT FEATURE AND COMPOSITION: THIS PRODUCT IS LAMINATED FROM MULTIPLE LAYERS OF THIN FABRIC

SIZE: ADULT

IMPLEMENTATION STANDARD: GB 2626-2006

DATE OF MANUFACTURING: SEE INNER BOX CERTIFICATE

VALIDITY: 3 YEARS

PACKING SPECIFICATION: 20 PCS/CASE

STORAGE: STORED IN A VENTILATED, DRY, HYGIENIC AND NON-CORROSIVE GAS PLACE.

CAUTION: PATIENTS WITH DIFFICULTY BREATHING OR CONGENITAL HEART DISEASE, PLEASE USE WITH CAUTION;

THIS PRODUCT CANNOT BE USED AS A GAS MASK IN A TOXIC ENVIRONMENT.





★ ★ ★ ★ ★  
KN95

# MASK

FACILITY

FLIGHTY

DEFEND

BREATHES



**FACCE MASK**



**KN95**



# KN95

## Wearing method

1. Face the mask without nose clip place the nose clip over the mask, hold the mask on your face, put the mask on your chin.
2. Just pull the earband to the outside of both ears and harit.
3. Put your fingers in the middle of the metal nose clip, press in while moving the fingertips to both sides along the nose clip, until the nose clip is fully pressed into the shape of the bridge of the nose.
4. Put your hands on your





# KN95

Wearing method

## 产品合格证 PRODUCT CERTIFICATE

产品名称: KN95防护口罩(非医用)  
Product Name: KN95 protective mask (Non-medical)

执行标准(Standards): GB2626-2006

数量(QTY): 10PCS

产品规格(Size): 160mm\*95mm (4 ply)

成份: 50%无纺布 + 25%熔喷布 + 25%热风棉  
Raw material: 50% non-woven fabric + 25% melt blown fabric + 25% cotton filtration fabric

产品批次(LOT): 20200418

生产日期(MER DATE): 2020-04-18

有效日期(EXP DATE): 2022-04-17

检验工号 (Inspector): 02

生产商: 汕头市海美思服饰有限公司

Manufacturer: shantou, himiss Clothing Co., Ltd

地址: 汕头市潮阳区和平镇老和惠路737号

Add: NO.737 of old Hehui Road Hebin Town, Chaoyang District, China



